



Additional Learning Needs

Fact Sheet

Loop Induction – LANSDOWNE CAMPUS

STUDENT RESPONSIBILITIES

What to do if you need to use the loop

1. Go to Additional Learning Needs Department (based in the Library at Talbot Campus room DLG17)
2. Discuss your needs with a member of staff who will loan you equipment to suit your needs.



- This could be either a red necklace or black headphone set.
- The red necklace is used in conjunction with the loop system and a hearing aid.
- The black headphone is used independently for people who do not have hearing aids but are hard of hearing.

Operating instructions

1. The Sennheiser symbol sign should be facing upwards.
2. Switch on and adjust the volume
3. If you are using the loop system you will need to set your hearing aid to T.

Useful Information

- You can negotiate loans for one off use or for the entire term. If you loan the equipment on a long term basis the batteries will need to be recharged regularly.
- It is a good idea to arrange a time to meet with your tutor before the planned lectures. You could use this opportunity to address any concerns you may have. The tutor will also become aware of your needs and can make preparations to use the induction loop.

Further information about the induction loops is available at:

http://www.bournemouth.ac.uk/ict_support/documents/131043.pdf

Further information

If you require any further information about supporting ALN students please contact:

Additional Learning Needs Service
Talbot Campus Library and Learning Centre
Academic Services
Talbot Campus
Bournemouth University
Fern Barrow
Poole
Dorset
BH12 5BB

Tel: 01202 965327

Fax: 01202 965844

E-mail: alnge@bournemouth.ac.uk

www.bournemouth.ac.uk/disabilities/